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Presentation

EN

Breastfeeding is one of the best experiences for a new mom, but there can be some obstacles and pains that can spoil this lovely moment. One of the most common obstacle are rhagades, small wounds that appear because of the rubbing between the palate and the tongue of the baby.

Thanks to **Koala Silver Cups**, the 999 silver cups, you can prevent and cure rhagades, protecting your breast from external agents and bacteria, and easing the pain in the first 24/48 of use.

Why to use Koala Silver Cups?

- The Koala Silver Cups are made of 999 silver and are nickel-free. For more than a century silver has been used in medicine to treat various types of skin diseases, in fact it is found that in the presence of moisture releases substances that have an

antibacterial and healing action.

- Their shape adheres perfectly to your breast, protecting it from any friction with clothes or external agents.
- The obvious savings both in economic and environmental terms. Once purchased the cups, if properly stored, they can be used by the same mother for future pregnancies, avoiding to purchase other products such as creams or single-use eliminators.

Warnings:

- If you are aware of any specific allergies to silver or if allergic reactions occur, stop using Koala Silver Cups immediately.
- Use only and exclusively on the area around the nipple.
- Respect the following instructions for use, washing and storage. Improper use or storage may

compromise product properties.

- The product does not contain NICHEL.

Instructions for use:

- When first used, wash the cups with lukewarm water and neutral and odorless soap, rinse and dry with a clean cloth. Repeat the washing operations each time they are used.
- Insert the cups between the breast and the bra, on the nipples.
- Before breastfeeding, gently remove the cups and wash the breasts with lukewarm water.
- At the end of breastfeeding apply a drop of milk or water inside the cup so that it can adhere perfectly to the breast.

Storage

- After having washed them, put the cups in their original packaging, inside their box. Keep the packaging in a cool, dry and clean place away from sources of heat or sunlight.
- We recommend keeping all the packaging during use, in case you have lost or thrown you can keep your cups in a clean cloth to avoid contact with the air and exposure to light that cause oxidation.

Best-before date: If all the guidelines have been followed to preserve the device, the product has no expiry date.

Koala Babycare suggests

- Start using the Koala Silver Cups 7-10 days before delivery as a preventive measure.
- Remember to always wash the cups and dry them

with a dry cloth to prevent oxidation of silver (blackening).

- If a large quantity of milk should come out, use the absorbent cups between **Koala Silver Cups** and the bra.
- If silver blackening occurs, you can wash the cups by soaking them in boiling water and baking soda (50 g per liter) until the stain has disappeared. If the stain should persist, write us an email to **info@koalababycare.com** and we will find a solution together.

Discover all the other tips for an enjoyable breastfeeding by downloading the complete manual through the QR Code that you find on the package, as a gift for you the extension of the KBC lifetime warranty.

Thank you for choosing Koala Babycare, for us your opinion is really important.

Koala Silver Cups met your expectations? Leave your review on our Amazon page and help many other mothers to choose their own silver cups! If you have experienced any problems, please send us an email to **info@koalababycare.com** and let us know so we can fix it together!

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